

# Roger Touchscreen Mic Speaker Microphone

Quick setup guide

roger

PHONAK  
life is on




1 Check the device is charged



2 Power on the Unit




> Press and Hold the Power button for 3 Seconds until the screen comes on

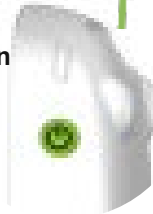

3 Place both Presenter Microphone and Receiver together (Must be within 1 cm). Press connect on the Touchscreen The Receiver unit will display a Tick



Max 10 cm

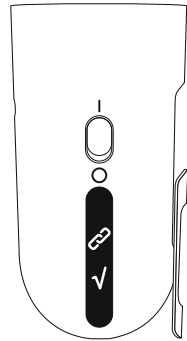

## Video Instruction

The QR below will link you to video instructions. Please scan this with a mobile phone




## Using Roger Touchscreen

1. **Lanyard Mode:** (Direct instruction)  
Presenter wears around neck within 6-8 inches from mouth




Lanyard

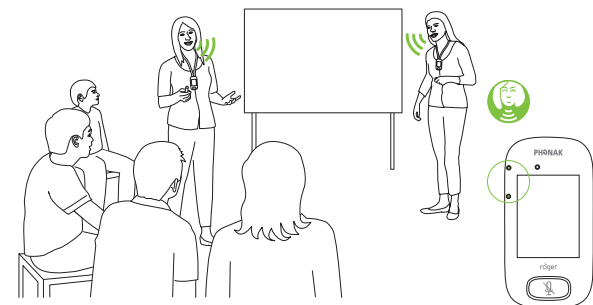

2. **Small Group:** (Small group work)  
Place flat on table to pick up voices of the group.



Small Group

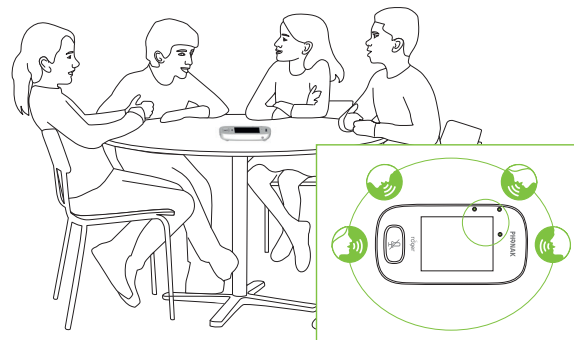

3. **Pointing:** (handheld use)

Speaker can point at the person speaking, within 10 feet away; good for class skits or presentations when multiple people are speaking.



Pointing



**Microphone Mode:**

The microphone will auto set.  
If needed you can force the microphone Mode

