

Roger NeckLoop Hearing Loop Reciever


Quick setup guide

roger




PHONAK
life is on


1 Check the device is charged

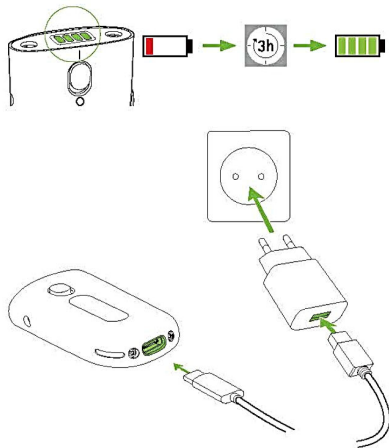

2 Attach the neck loop

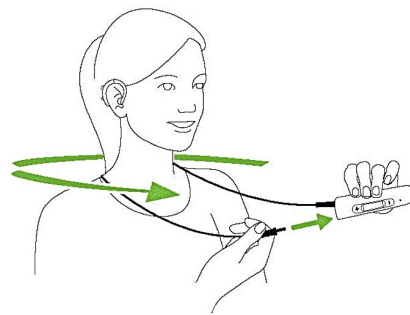


3 Switch on

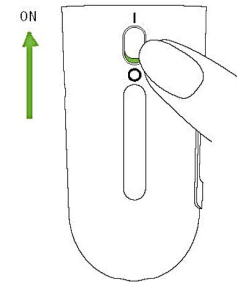

4 Activate T-coil program in hearing aid or Plug in supplied Headphones




Option 1: Hearing Aid

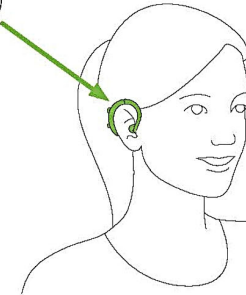


Option 2: Headphones




5 Switch on the Roger microphone

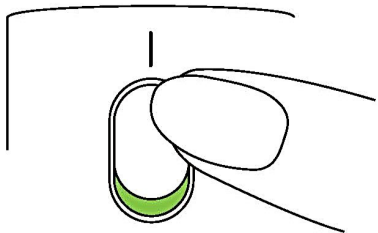


6 Place both Presenter Microphone and Receiver together. Press connect

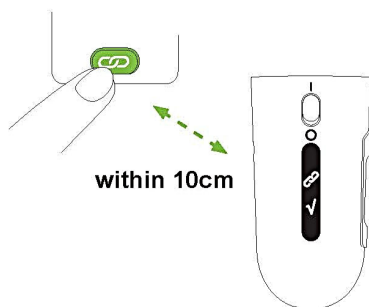

7 Speech is transmitted from connected microphones

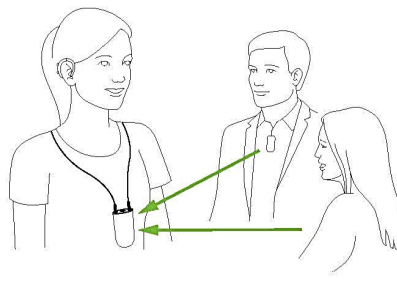

8 Adjust volume (Side)

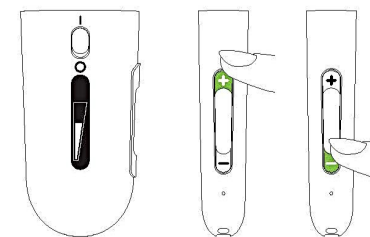

Video Instruction

The QR below will link you to video instructions. Please scan this with a mobile phone



sonova
HEAR THE WORLD

